Kaleido Controller

EN – User Manual

This User Manual provides instructions for operating the Kaleido Controller and applies exclusively to this device. For guidance on the medical use of the authorised application, please refer to the User Manual supplied with that application. For instructions on operating the Kaleido Insulin Pump or a compatible CGM, please refer to the User Manual provided with the relevant device.

Important notice

Before using the Kaleido Controller, read the information in this User Manual. You may also find this user manual online at: www.hellokaleido.com. Retain this documentation for future reference.

Intended purpose

The Kaleido Controller is a smartphone configured under Mobile Device Management (MDM) to operate in a locked-down mode as a Class I medical device accessory. Its intended purpose is to host and enable the execution of an authorised medical device software in a controlled environment, providing connectivity, security, and usability functions necessary for safe and reliable use. The Kaleido Controller itself does not perform measurements, calculate therapy, determine dosing, interpret data or provide clinical decision support. The device is intended to be used in a non-clinical environment.

Contraindications

There are no known contraindications for the use of the Kaleido Controller.

Technical specifications

Product Name: Kaleido Controller

Wireless communication with the Kaleido Insulin Pump and compatible CGM: Bluetooth BLE

Supported operating system (OS): Refer to the medical device application's user manual for the supported OS and versions

Refer to the smartphone user manual for the complete technical specification of the device.

Accessing device settings

The Kaleido Controller operates with a restricted Android user interface. As a result, device settings cannot be accessed via the usual Android method.

To access the device settings:

1. Locate the Info button (ⓘ) at the bottom of the screen.
2. Press the Info button.
3. On the next window, press the Settings button to access the available device settings.

Note: Only the settings visible and accessible are meant to be used on the Kaleido Controller. Do not attempt to modify settings outside of those made available on the user interface, as doing so may compromise the integrity and safe operation of the device.

Security

Access to the Kaleido Controller is protected by an automatic screen lock after 2 minutes of inactivity to protect device integrity and patient data.

A primary security method (passcode/PIN, or pattern) is enforced by the Kaleido Controller. Where available, a biometric authentication method (e.g. fingerprint or face recognition) can be added so that the Kaleido Controller can be unlocked more easily when needed.

General notes & instructions

The Kaleido Controller is delivered with a screen protector pre-applied and a protective case included in the packaging. These accessories are general consumer items and do not affect the safety or performance of the device.

If a low-battery indication is shown, charge the device as soon as possible. Charge the Kaleido Controller according to the smartphone manufacturer's user manual. Power adapter is not included. Charge only from a compatible USB Type-C power source.

If the Kaleido Controller indicates loss of connectivity, promptly restore the connection by bringing the devices within range and confirming that connectivity has been re-established.

Wi-Fi access is required to complete the setup of the Kaleido Controller. Ensure a Wi-Fi network or mobile hotspot is available before starting.

Precautions & Warnings

WARNING

Do not use the Kaleido Controller if the device is visibly damaged (e.g. cracked screen). Stop using the device and contact your distributor or Kaleido Care.

WARNING

The Kaleido Controller shall only be operated by the intended user or by a caregiver acting on behalf of the intended user.

WARNING

Loss of connectivity between the Kaleido Controller and the insulin infusion system may delay therapy adjustment or prevent timely review of system status.

PRECAUTION

Keep the Kaleido Controller charged to a level sufficient for the expected period of use.

PRECAUTION

The Kaleido Controller is the dedicated platform for interaction with the authorized medical application and the insulin infusion system. Keep the Kaleido Controller available during therapy.

PRECAUTION

Failure to properly decommission the Kaleido Controller may result in unauthorized access to personal or therapy-related data.

Contact Kaleido Care

Telephone: + 31 (0) 800 19 10
Email: hello@hellokaleido.com
Website: www.hellokaleido.com

Report any serious incident that has occurred in relation to the Kaleido Controller to ViCentra and to the competent authority in the country in which you are established.

Disposal

When it's time to dispose of your Kaleido Controller, please contact us and return it to the following address for proper decommissioning.

ViCentra B.V.
Rijnzathe 6
3454 PV Utrecht
The Netherlands

Symbols

	CE Marking		Medical Device
	Manufacturer		Unique Device Identifier
	Date of manufacture		Single patient multiple use
	Consult instructions for use		Keep dry
	Serial Number		Waste Electrical and Electronic Equipment (WEEE)
	Catalogue number		Temperature limit
	Quantity		Do not use if package is damaged

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